



Caution after implant surgery

OSSTEM[®]
IMPLANT

Implant Treatment Procedure



Cautions after Implant Treatment



Bleeding

Keep the gauge in the mouth for two hours or more, and swallow any saliva or blood rather than spit it out.



Prescription

Must take the prescribed drug to alleviate pain and prevent infection.



Swelling

Repeatedly apply and remove an ice pack in 5 minutes interval. (It's not clear from this whether the ice pack should remain applied for a full five minutes, or just for a moment but done every five minutes.)



Alcohol, cigarette

Must refrain from alcohol and cigarettes for at least one month.



Food

Consume soft or liquid food for the 1st week after the surgery. refrain from consuming any tough or hard food.



Brushing

Rinse with mouthwash (antiseptic solution) twice a day, and avoid the site of treatment when brushing for one week after surgery.



Bath and rest

Avoid hot baths, hot packs, and intense exercise for 2-3 days after the surgery, and take a rest.

Contact the doctor immediately if you experience any abnormality after the surgery.